GUIDE D'UTILISATION / USER GUIDE

PortalMouse 2.2

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2. User guide – English

2.1. Introduction of PortalMouse

PortalMouse is a freeware created by François PRATS on November 2016, 3rd year occupational therapist student at Hyères les Palmiers OT School, in training on the **Plate-Formes Nouvelles Technologies** of **Raymond Poincaré Hospital (Assistance Publique des Hôpitaux de Paris)**.

This software's purpose is to help people with the use of the mouse for a joystick with two or three axis directions. **PortalMouse** was developed for computers with **Windows 7** or higher installed.

PortalMouse allows the mouse cursor to be screen border free and to reappear cyclically on the other side.

The software was developed in order to solve a problem encountered in clinical practice following the use of communication software based on pictograms with an occipital control on an electric wheelchair. This software prevented tedious manipulations of the control when moving the mouse in all four directions.

2.2. How to install PortalMouse

PortalMouse is an executable and requires no installation or administrator rights.

Hardware and software requirements :

- 32 or 64 bit Windows 7 or higher
- Only one screen is needed

PortalMouse currently supports all screen sizes and resolutions.

The software is automatically launched in **English** or **French** depending on the version of Windows used.


2.3. How to use PortalMouse

Once the executable is downloaded and saved to the computer's hard disk, run **PortalMouse**.


A window opens and activates the **PortalMouse** mode :


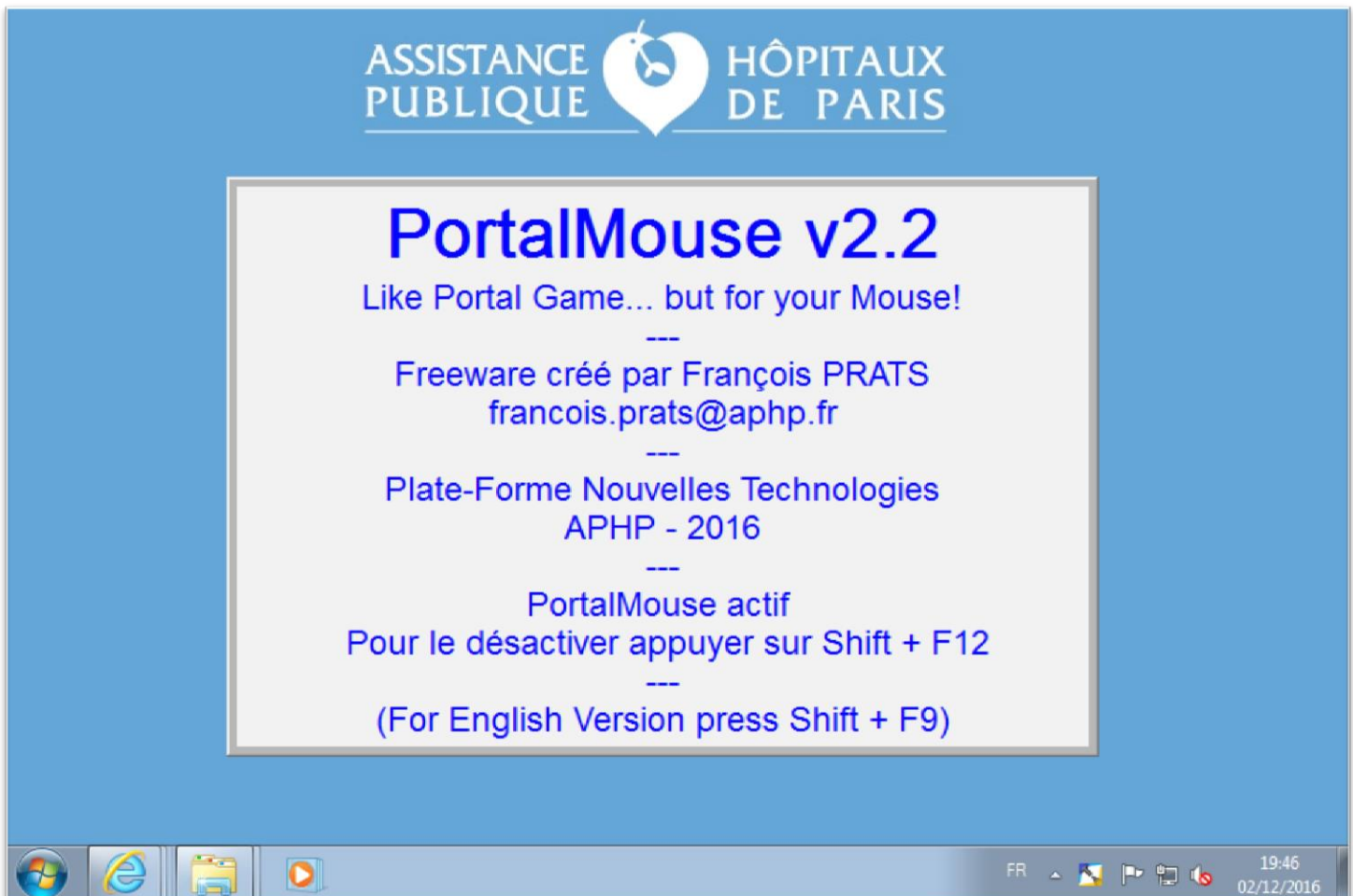
Border restrictions are now removed. Moving to a corner of the screen allows you to "teleport" the mouse anywhere on the screen :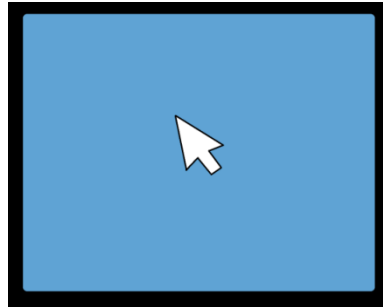

It is now possible to make cyclical movements on the horizontal and / or the vertical scale of the screen.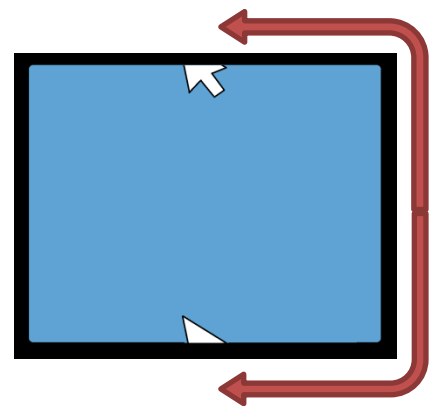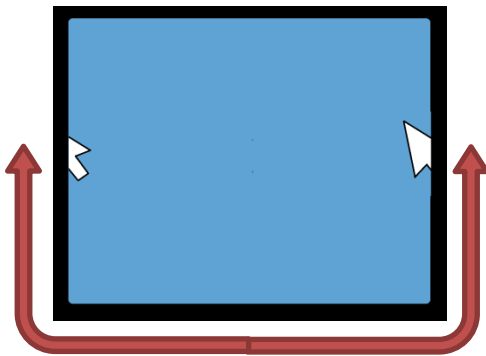

This **PortalMouse** feature can be enabled or disabled by pressing both **SHIFT + F12** keys.

PortalMouse can be launched as the session starts by adding it on startup.

PortalMouse is available in English and French. To change the language press both **SHIFT + F9** keys.

To stop the software press both **SHIFT + F12** keys to disable the **PortalMouse**, then press **F4** key.

To remove the software completely, stop the software then delete the executable.

If you have any further inquiries or comments about **PortalMouse** please contact via e-mail: francois.prats@aphp.fr